

hCG (Pregnancy Test)

PH - 100c & PH - 100s

Human chorionic gonadotropin (hCG) is a glycoprotein hormone produced by the developing placenta shortly after fertilisation. In normal pregnancy, hCG can be detected in both urine and serum as 7 to 10 days after conception during early gestational growth, make it an excellent marker for the early detection of pregnancy. The hCG Rapid Test is a chromatographic immunoassay for the qualitative detection of Human chorionic gonadotropin (hCG) in urine or serum to aid in early of detection of pregnancy.

Specification :

- Sample : Urine
- Volume : 3 drops
- Reading time : 5 - 30 minutes
- Sensitivity : 25 mIU / mL
- Long expiry date
- Package : 25 tests or 50 tests

Procedure :

- Add 3 drops of sample into sample well.
- Read the result within 30 minutes after adding sample.

Interpretation of Result

Positive

Negative

Invalid